EXPECTED VALUES

Method	CONTROL 1	CONTROL 2
Hemoglobin S	Negative	Positive

INTENDED USE

Sickle QC is intended to be used as a sickle cell control in testing for the presence of Hemoglobin S in solubility tests.

SUMMARY AND PRINCIPLE

It is an established laboratory practice to use a stable control to monitor the performance of diagnostic tests. This control is composed of stable materials that provide a means of monitoring the presence of sickling hemoglobins in solubility tests. It is sampled in the same manner as a patient specimen.

REAGENTS

Sickle QC is an *in vitro* diagnostic reagent composed of human erythrocytes suspended in a plasma-like fluid with preservatives.

PRECAUTION


Sickle QC is intended for *in vitro* diagnostic use only by trained personnel.

WARNING:

POTENTIAL BIOHAZARDOUS MATERIAL.

For *in vitro* diagnostic use. Each human donor/unit used in the preparation of this product has been tested by a FDA licensed method/test and found to be negative or non-reactive for the presence of HBsAg, Anti-HCV, NAT testing for HIV-1, HCV (RNA) and HIV-1/2. Each unit is also negative by a serological test for Syphilis (RPR or STS). Because no test method can offer complete assurance that infectious agents are absent, this material should be handled as potentially infectious. When handling or disposing of vials follow precautions for patient specimens as specified in the OSHA Bloodborne Pathogen Rule (29 CFR Part 1910, 1030) or other equivalent biosafety procedures.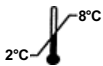

STABILITY AND STORAGE

Store Sickle QC upright at 2 - 8° C (35 - 46 F) when not in use. **Protect vials from overheating and freezing.** Unopened vials are stable through the expiration date. Opened vials are stable for 100 days, provided they are handled properly.

INDICATIONS OF DETERIORATION

After mixing, product should be similar in appearance to fresh whole blood. In unmixed vials, the supernatant fluid is expected to be dark in color; this is normal and does not indicate deterioration. Unacceptable results may indicate deterioration. **Do not use the product if deterioration is suspected.**

INSTRUCTIONS FOR USE

- Remove vials from the refrigerator and allow to warm to room temperature (15 - 30°C or 59 - 86°F) for 15 minutes before mixing.
- To mix, hold a vial horizontally between the palms of the hands. **Do not pre-mix on a mechanical mixer.**
 - Roll the vial back and forth for 20 - 30 seconds; occasionally invert the vial. Mix vigorously, but do not shake.
 - Continue to mix in this manner until the red cells are completely suspended. Vials stored for a long time may require extra mixing.
 - Gently invert the vial 8 - 10 times immediately before sampling.
- Analyze the sample as instructed in the reagent test kit instructions.
- After sampling:
 - Clean residual material from the cap and vial rim with a lint-free tissue. Replace the cap tightly.
 - Return vials to refrigerator within 30 minutes of use.

LIMITATIONS

The performance of this product is assured only if it is properly stored and used as described in this insert. Incomplete mixing of a vial prior to use invalidates both the sample withdrawn and any remaining material in the vial.

EXPECTED RESULTS

Sickle QC is designed to be handled in the same manner as a patient specimen. Analyze the sample as instructed in the reagent test kit. The positive control will appear as a cloudy turbid suspension as described in the reagent test kit instructions for use. The negative control will appear as a nearly transparent solution as described in the reagent test kit instructions for use.

TECHNICAL ASSISTANCE AND CUSTOMER SERVICE

For technical assistance or additional information, please call your dealer or local distributor. If there is no, you may call Bio-technne® Technical Service at 33 (0)2 99 35 19 36.

All brands and products are trademarks or registered trademarks of their respective companies.

R&D Systems, Inc.
614 McKinley Place NE
Minneapolis, MN USA 55413

AI5125-004 Rev 09/13

Bio-technne®
19 Rue Louis Delourmel
35230 Noyal Châtillon / Seiche
France

ISO5C00-V04 09/2019

RESULTATS ATTENDUS

Method	CONTROL 1	CONTROL 2
Hemoglobin S	Negative	Positive

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